


Immediate Implant Bridge

Clinical Preparation (First Appointment)

1. Take initial impressions and duplicate an existing denture

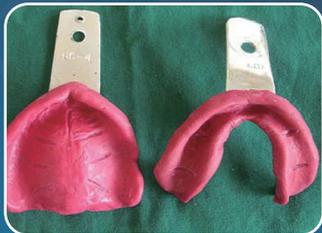

Take the upper and lower impression



Take another set of impressions with the existing dentures in the mouth and the occlusal bite. (A facebow record is recommended for greater accuracy)



Duplicate an existing denture with putty in the clinic or send it to our laboratory for duplication. (We will return it to the clinic on the same day)

2. Take patient photos for reference



Take photos are recommended for patient's tooth relationship: smile line, tooth position and occlusion.

3. Fill-in the Modern Premium Line laboratory form

CODE: _____ (M/F)
 AGE: _____ (M/F)
 DATE REQUIRED: _____

MODERN Dental Laboratory

Crown/ Bridge Maryland Bridge/ RBB 1 wing* 2 wings Inlay/ Onlay Veneer Implant Restoration

System: _____

Shade Matching
 Central
 Lai Chi Kok

Internal Use
 Technicians: _____
 Material Log: _____

Tooth Position

18	17	16	15	14	13	12	11	21	22	23	24	25	26	27	28
47	46	45	44	43	42	41	31	32	33	34	35	36	37	38	39

Acrylic Teeth
 Standard Premium* Heavy Light* No

Metal Design
 Procera Zr Ti

Embrasure
 Open Close

Invoiced by: _____
 Conversation Log: _____

Indicate the desired implant positions on our lab sheet and provide the shade or any requests

Modern Dental Laboratory will provide the following items for clinical surgery placement in 7 working days



An acrylic positioning stent



A hollowed-out full temporary denture



An arch form wire for splinting impression coping




A custom open tray


Clinical surgery placement of the implants (Second Appointment)




Acrylic positioning stent

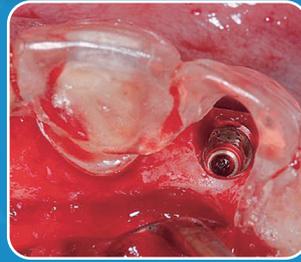

Place the implants between the window(s) opened in the acrylic positioning stent

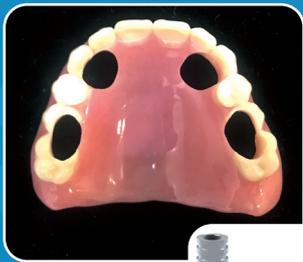


Hollowed-out full temporary denture + temporary cylinders

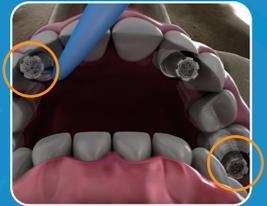

1. Place the temporary cylinders through the temporary denture
2. Direct chairside reline on the fitting surface of the temporary denture
3. Use cold cure resin to pick up at least two temporary cylinders (at least one from each quadrant)
4. Cut off excess temporary cylinders and take the occlusal bite



An arch form wire + impression coping + a custom open tray

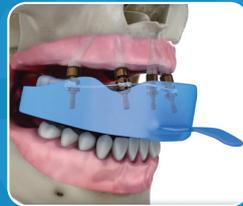

1. Attach impression coping on the abutments
2. Attach the provided arch wire onto the multiple copings using light cure material or pattern resin to avoid any movement by splinting
3. Use the open tray provided and PVS material to take impression

Modern Dental Laboratory convert your temporary denture to an **Immediate Implant Bridge** (24 hrs)



Modern Dental Laboratory Company Limited

17/F, CEO Tower, 77 Wing Hong Street, Cheung Sha Wan, Kowloon, Hong Kong

Tel: +852 3766 0777

www.moderndentallab.com

Stock Code: 3600

©2020 Modern Dental Laboratory Company Limited. All rights reserved.
CBFC01E4V2

